Quick Start Guide

Schnellstartanleitung

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Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands

Huawei Device Co., Ltd.

No.2 of Xincheng Road, Songshan Lake Zone, Dongguan, 523808 Guangdong,

P.R.China

Issue Date: 2022-08

MLY-B10

MLY-B11

Quick Start Guide

0. Safe Use

This device is a household product and is suitable for wrist blood pressure measurement for adults aged 18 and above.

If you use a defibrillator while wearing this wrist blood pressure monitor, the blood pressure measurement function of this watch cannot be guaranteed or maintained.

If you have any questions, please contact the local customer service hotline. For details, please refer to <https://consumer.huawei.com/en/support/hotline/>.

Warning

1. NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician. ONLY a physician is qualified to diagnose and treat hypertension.
2. Do not take blood pressure measurements too often as bruising may occur due to blood flow interference.
3. Do not use this device on a wound or inflamed arm or wrist as this may cause further injury.
4. Do not wear this device when receiving intravenous infusions or blood transfusions, or when having a catheter inserted through the wrist, as the device may interfere with blood flow and cause injury.
5. Do not use this device if you are allergic to nylon, fluororubber, or thermoplastic polyurethane elastomer (TPU).
6. Consult with your physician before using this device if you have had a mastectomy or lymph node clearance.
7. Use of this device may cause the temporary loss of function of the monitoring medical electrical (ME) equipment used simultaneously on the same limb.
8. Consult with your physician before using this device if you have prior arterial obstructive disease, arteritis, blood circulation disorders, or blood diseases.
9. Observe your arm during the measurement to ensure that this device is not affecting the blood flow.
10. Do not use this device on infants, toddlers, children, or persons who cannot express themselves.
11. Keep the charging cradle (with the power cable) away from infants, toddlers, or children to avoid the

risk of strangulation.

12. Do not use this device in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, and computerized tomography (CT) scanners.

 **Caution**

1. If the wrist circumference exceeds the supported range (13.0—20.0 cm), the accuracy of blood pressure measurement may be affected due to insufficient verification.
2. NOTE Patient moving, trembling, or shivering may affect the blood pressure measurement reading.
3. This device has not been validated for use on pregnant patients.
4. Portable and RF communication devices may affect this device. Do not place the device near or on top of other devices. Otherwise, the radiation may increase and the anti-interference capability may decrease.
5. If the measured blood pressure is abnormal, you can take a measurement again 5 minutes later or consult with your physician.
6. This device may cause side effects such as allergies. In this case, stop wearing it and consult with your physician.
7. Consult with your physician before using this device if you have arrhythmias (such as atrial or ventricular premature beats, atrial fibrillation, atrial flutter, ventricular flutter, ventricular fibrillation, ventricular tachycardia, borderline tachycardia, sinus arrest, pathological sinus syndrome, and pacemaker rhythm).
8. Consult with your physician before using this device if you have arterial sclerosis, poor perfusion, diabetes, pre-eclampsia, or renal disease.
9. The accuracy of this blood pressure monitor has been carefully tested and is designed to last for the full service life. Once the device exceeds its expected service life, it may no longer meet the advertised performance capabilities.
10. Do not disassemble or attempt to repair this device by yourself.
11. The user must be an intended operator.

1. Product Information

1.1 Intended Use

The Wrist Blood Pressure Monitor is a digital self-monitor intended for use in measuring blood pressure and pulse rate in adult patient population with wrist circumference ranging from 13.0 cm to 20.0 cm (5.12–7.87 inches).

1.2 Contraindications

No contraindications have been found.

1.3 Clinical Benefit

This product helps users perform blood pressure self-monitoring, and assists users who have been diagnosed with hypertension in identifying episodes of intermittent hypertension during daily activities.

1.4 Performance Indicators

General Specifications

Name	Wrist Blood Pressure Monitor
Models	MLY-B10, MLY-B11
Type of Protection Against Electric Shock	Internal power supply
Degree of Protection Against Electric Shock	Type BF (Airbag, the bottom of the device)
Level of Protection Against Harmful Liquids and Particles Entering Inside	IP68 (This device complies with the IP68-rated degree of protection under the IEC 60529 standard. It is dust-tight, and its enclosure can be continuously immersed in 1.5 meter-deep water for 30 minutes, without causing any harmful effects.)
Level of Safety Against Flammable Anesthetic Gas Mixed with Air, Oxygen, or Nitrous Oxide	Non-AP/APG type
Running Mode	Continuous running
Power Supply	Rechargeable Li-ion Polymer Battery (Rated capacity: 451 mAh/1.74 Wh; rated voltage: 3.87 V)

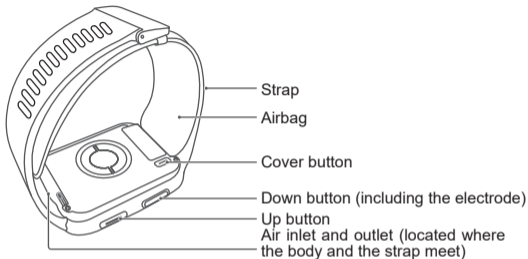
Battery Life Information	<p>Typical usage: 7 days</p> <p>Typical battery life: A new battery on a full charge will last for approximately 500 cycles when operating at room temperature (25°C).</p> <p>The battery level can be seen on the device screen.</p>
Durable Period (Service Life)	Device body: 3 years; airbag & strap: 1 year. (Product Service life is normally considered as the time from first use until the device ceases to fulfil its intended use.)

Blood Pressure Measurement

Measuring Method	Oscillometric method
Measurement Range	<p>Pressure: 0–300 mmHg (0–40 kPa)</p> <p>Pulse rate: 40–180 times/min</p> <p>Diastolic blood pressure: 40–130 mmHg (5.3–17.3 kPa)</p> <p>Systolic blood pressure: 60–230 mmHg (8–30.7 kPa)</p>
Accuracy	<p>Pressure: ± 3 mmHg (± 0.4 kPa)</p> <p>Pulse rate: $\pm 5\%$</p>
Inflation	Automatic pressure pump
Deflation	Automatic fast deflation
Measurable Wrist Circumference	<p>M-sized airbag: 13.0–16.0 cm/5.12–6.30 in</p> <p>L-sized airbag: 16.1–20.0 cm/6.34–7.87 in</p> <p>Please select the appropriate airbag according to your measured wrist circumference.</p>
Maximum temperature of the applied part	Lower than +43°C (Non-Charging Mode)

This device complies with IEC 80601-2-30, and has been clinically investigated according to the requirements of ISO 81060-2:2018/A1: 2020. In the clinical validation study, K5 was used on more than 85 subjects for determination of diastolic and systolic blood pressure.

1.5 Structure Diagram and Packing List

This product consists of a device body, a cuff (strap), and a wireless charging cradle. For its materials, see the label on the package.

Packing list:

- Device (with an L-sized strap and an L-sized airbag) x 1
- Independent cover x 1
- Quick Start Guide x 1
- Warranty Card x 1
- M-sized airbag x 1
- L-sized and M-sized airbag clothings (spare) x 1
- Wearing Guide x 1
- M-sized strap (without the clasp) x 1
- Wrist circumference ruler x 1
- Charging cradle (with the power cable) x 1

2. Blood Pressure Measurement

2.1 Principles

This device measures blood pressure by using the oscillometric method. As the strap (airbag) pressure increases, the device extracts and analyzes the signals of pressure oscillations in the airbag using the pressure sensor, and automatically calculates the high pressure (systolic pressure), low pressure (diastolic pressure), and pulse rate.

2.2 Working Environment

Network Environment	Bluetooth 4.2 or later
Temperature	5°C to 40°C
Humidity	10% RH to 95% RH (excluding condensation)
Atmospheric pressure	80 kPa to 105 kPa

Caution

- Keep the device away from electromagnetic interference, vibration, and noise.
- Failure to comply with the working environment requirements specified in this guide will cause the product to fail to reach the claimed performance specifications or damage the parts.

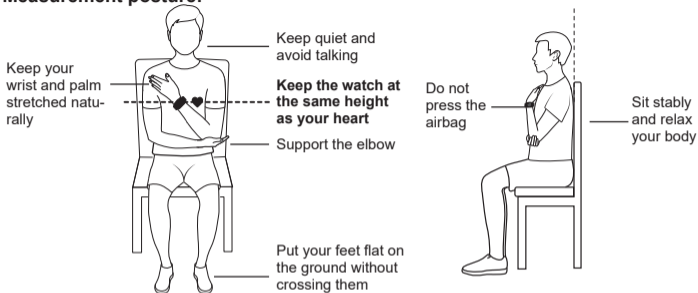
2.3 Measurement Requirements

Any blood pressure measurement data may be influenced by the location of the measurement, posture (standing, sitting, or lying down), movement, or physiological conditions.

To ensure the accuracy of the measurements, observe the following requirements:

- Rest for 5 minutes before a measurement and keep your body naturally relaxed until the measurement is complete. Do not perform the measurement when you feel emotional or stressed.
- Perform a measurement in a quiet environment. Do not speak, bend your fingers, or move your body or arms during the measurement.
- Rest your wrist for 1 to 2 minutes or longer before you perform the next measurement. During this period of time your arteries will return to how they were before the blood pressure was measured.
- Avoid eating, drinking alcohol, smoking, and exercising for at least 30 minutes before taking a measurement.

Measurement posture:

- Remove the device if it does not start deflating during a blood pressure measurement.
- Press and hold the Up button for more than 15 seconds to restart the device.

Description of GUI abbreviations

No.	Abbreviation	Full Name
1	SYS	SYSTOLIC BLOOD PRESSURE
2	DIA	DIASTOLIC BLOOD PRESSURE
3	cm	centimeter
4	in	inch

2.4 Entering the Static Mode

This feature is mainly for professionals to enter the static pressure test mode of this device, and to use the standard pressure gauge to test its static performance.

3. Storage and Maintenance

Caution

- The maintenance of this device only involves the replacement of accessories (the airbag, airbag clothing, and strap).
- The accuracy of the pressure sensor in this device has been strictly tested, so you do not need to check it by yourself.

3.1 Storage Requirements

Storage and transportation conditions:

Temperature: -20°C to $+55^{\circ}\text{C}$

Humidity: 10% RH to 95% RH (excluding condensation)

Atmospheric pressure: 80 kPa to 105 kPa

Do not store the device in the following environments:

- Environments with water, high temperatures, dampness, direct sunlight, dust, or salt.
- Environments that can generate tilts, vibrations, or impacts.
- Environments where chemicals are stored or corrosive gases are generated.

It is recommended that you put the airbag in a box or storage bag when you are not using it.

Do not put the device in places that are exposed to direct sunlight, such as on a car dashboard or on a window sill.

Keep the device away from heat sources or exposed fire sources, such as electric heaters, microwaves, ovens, water heaters, fireplaces, candles, and other places that may generate high temperatures.

To prevent injuries, malfunctions, or explosions, do not allow children or pets to swallow or bite the device or its accessories.

3.2 Transportation Conditions

Avoid strong impacts, direct collisions, and exposure to sunlight or rain during transportation.

3.3 Accessory Replacement

For details, please see the Wearing Guide.

Note: Replacing the original parts with third-party parts may cause measurement errors.

3.4 Cleaning

Keep the product clean and dry during daily use.

1. It is recommended that you use a soft cloth or towel dipped in clean water to clean the device body, and then use a dry cloth to wipe it down.
2. It is recommended that you use a soft cloth or a tissue dipped in clean water to clean the strap and the airbag, and then use a dry cloth to wipe them down, or let them air-dry before wearing them.
3. It is recommended that you use mild detergent or soap to clean the airbag clothing, and let it dry before wearing it.

Caution

- When cleaning the device body or airbag, try to prevent any liquid from entering the device.
- Do not immerse the device or airbag in water, and do not dry the airbag using a hairdryer.
- Do not use organic solvents (such as hand sanitizer, soapy water, or other acidic solvents), or strong detergents or disinfectants to clean the device body, the strap, or the airbag.
- Do not use sharp or hard objects (including stainless steel scouring balls, or abrasive sponges) to clean the device.

4. Explanations of Symbols and Figures

Figures and Symbols	Description
	Type BF Applied Part
	Caution
IP68	Degree of protection of the enclosure
	Main chip trademark
	Indicates that the product is capable of being recycled.
 Bluetooth	Short-distance radio technology, Bluetooth certification mark.
	Forest Stewardship Council
	Recycle the object when it is discarded.
	Non-ionizing radiation
MD	Medical device
CE 0197	CE mark. Indicates that the device complies with EU 2017/745.
	The operation guide must be read.
	Recyclable non-corrugated fiberboard (board)
EC REP	Authorized representative in the European Union
	Manufacturer
	Remove the packaging and dispose of it in the appropriate recycling bin according to local laws and regulations.
	The range of temperature to which the medical device can be safely exposed.
	The range of humidity to which the medical device can be safely exposed.
	The range of atmospheric pressure to which the medical device can be safely exposed.
SN	Serial number
	Importer
#	Model number
UDI	Unique device identifier
	Date of manufacture
	Correct wearing position

Note: The two digits after the 12-digit characters on the back of the product indicate the manufacturing year and month. The first digit is the last digit of the year, the second digit indicates the month, and the months from 1 to 12 are represented by 1 to 9, A, B, and C respectively. For example, 13 corresponds to March, 21.

5. Declaration of Electromagnetic Compatibility (EMC)

MLY-B10/MLY-B11 manufactured by Huawei Device Co., Ltd. conforms to IEC 60601-1-2:2014/AMD1:2020 Electromagnetic Compatibility (EMC) standard.

Further documentation in accordance with this EMC standard is available at <https://consumer.huawei.com/en/>.

Safety information

Copyright © Huawei 2022. All rights reserved.

THIS DOCUMENT IS FOR INFORMATION PURPOSE ONLY, AND DOES NOT CONSTITUTE ANY KIND OF WARRANTIES.

Trademarks and Permissions

The *Bluetooth*[®] word mark and logos are registered trademarks owned by *Bluetooth SIG, Inc.* and any use of such marks by Huawei Technologies Co., Ltd. is under license. Huawei Device Co., Ltd. is an affiliate of Huawei Technologies Co., Ltd.

Other trademarks, product, service and company names mentioned may be the property of their respective owners.

Privacy Policy

To better understand how we protect your personal information, please see the privacy policy at <https://consumer.huawei.com/privacy-policy>.

Blasting Caps and Areas

Turn off your mobile phone or wireless device when in a blasting area or in areas posted turn off “two-way radios” or “electronic devices” to avoid interfering with blasting operations.

Petrol stations and explosive atmospheres

In locations with potentially explosive atmospheres, obey all posted signs to turn off wireless devices such as your phone or other radio equipment. Areas with potentially explosive atmospheres include fuelling areas, below decks on boats, fuel or chemical transfer or storage facilities, areas where the air contains chemicals or particles, such as grain, dust, or metal powders.

Operation and safety

- Using an unapproved or incompatible power adapter, charger or battery may cause fire, explosion or other hazards.
- Ideal operating temperatures are 5°C to 40°C.
- Avoid placing metal objects on the surface of the wireless charging cradle, as they could cause the equipment to heat up and malfunction. Only use the wireless charging cradle in environments with an ambient temperature ranging from 5°C to 40°C.
- If the ambient temperature is too high, the wireless charger may stop working. If this occurs, you are advised to stop wireless charging.
- To keep the wireless charger in good condition, avoid using it inside a vehicle.
- Adult supervision is advised if children are to use or come into contact with this product.
- Please consult your doctor and the device manufacturer to determine if operation of your device may interfere with the operation of your medical device.
- Switch off your wireless device when requested to do so in hospitals, clinics or health care facilities. These requests are designed to prevent possible interference with sensitive medical equipment.
- Some wireless devices may affect the performance of hearing aids or pacemakers. Consult your service provider for more information.
- Pacemaker manufacturers recommend that a minimum distance of 15 cm be maintained between a device and a pacemaker to prevent potential interference with the pacemaker. If you are using a pacemaker, hold the device on the side opposite the pacemaker and do not carry the device in your front pocket.
- Keep the device and the battery away from excessive heat and direct sunlight. Do not place them on or in heating devices, such as microwave ovens, stoves, or radiators.
- Observe local laws and regulations while using the device. To reduce the risk of accidents, do not use your wireless device while driving.
- To prevent damage to your device's parts or internal circuits, do not use it in dusty, smoky, damp or dirty environments or near magnetic fields.
- Do not use, store or transport the device where flammables or explosives are stored (in a gas station, oil depot, or chemical plant, for example). Using your device in these environments increases the risk of

explosion or fire.

- Dispose of this device, the battery, and accessories according to local regulations. They should not be disposed of in normal household waste. Improper battery use may lead to fire, explosion, or other hazards.
- The device has undergone testing and has demonstrated water resistance in certain environments.
- The device has a built-in, non-removable battery. Do not attempt to remove the battery, otherwise the device may be damaged.
- If the battery leaks, ensure that the electrolyte does not make direct contact with your skin or eyes. If the electrolyte touches your skin or splashes into your eyes, immediately flush with clean water and consult a doctor.
- When charging the device, make sure the power adapter is plugged into a socket near the devices and is easily accessible. Unplug the charger from the electric socket and from the device when the device is not used for a long time.
- Unplug the charger from electrical outlets and the device when not in use.
- Do not place sharp metal objects, such as pins, near the microphone. The microphone may attract these objects and result in injury.
- Wireless devices may interfere with the airplane's flight system. Do not use your device where wireless devices are not allowed according to the airplane company's regulations.
- To ensure the best audio performance of your device's microphone, use a cloth to wipe the area of the microphone which has been exposed to water. Do not use the microphone until the wet areas are totally dry.
- Switch off your wireless device whenever you are instructed to do so by airport or airline staff. Consult the airline staff about the use of wireless devices on board the aircraft, if your device offers a "flight mode", this must be enabled prior to boarding an aircraft.
- Ensure that the power adapter meets the requirements of IEC/EN 62368-1 and has been tested and approved according to national or local standards.
- Protect the wearable and its accessories from strong impact, strong vibration, scratches, and sharp objects, as these may damage the device.
- Make sure that the wearable and its accessories stay clean and dry.

- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.

Disposal and recycling information

The crossed-out wheeled-bin symbol on your product, battery, literature or packaging reminds you that all electronic products and batteries must be taken to separate waste collection points at the end of their working lives; they must not be disposed of in the normal waste stream with household garbage. It is the responsibility of the user to dispose of the equipment using a designated collection point or service for separate recycling of waste electrical and electronic equipment (WEEE) and batteries according to local laws.

Proper collection and recycling of your equipment helps ensure electrical and electronic equipment (EEE) waste is recycled in a manner that conserves valuable materials and protects human health and the environment, improper handling, accidental breakage, damage, and/or improper recycling at the end of its life may be harmful for health and environment. For more information about where and how to drop off your EEE waste, please contact your local authorities, retailer or household waste disposal service or visit the website <https://consumer.huawei.com/en/>.

Reduction of hazardous substances

This device and any electrical accessories are compliant with local applicable rules on the restriction of the use of certain hazardous substances in electrical and electronic equipment, such as EU REACH, RoHS and Batteries (where included) regulations, etc. For declarations of conformity about REACH and RoHS, please visit our website <https://consumer.huawei.com/certification>.

RF exposure requirements

Your device is a low-power radio transmitter and receiver. As recommended by international guidelines, the device is designed not to exceed the limits established by the European Commission for exposure to radio waves.

EU regulatory conformance

Hereby, Huawei Device Co., Ltd. declares that this device, MLY-B10/MLY-B11, is in compliance with the RED Directive 2014/53/EU, RoHS Directive 2011/65/EU, and MDR Regulation 2017/745/EU. The full text of

the EU declaration of conformity, and most recent information about accessories & software are available at <https://consumer.huawei.com/certification>.

Frequency bands and power

(a) Frequency bands in which the radio equipment operates: Some bands may not be available in all countries or all areas. Please contact the local carrier for more details.

(b) Maximum radio-frequency power transmitted in the frequency bands in which the radio equipment operates: The maximum power for all bands is less than the highest limit value specified in the related Harmonized Standard.

When charging the device, ensure it is 20 cm away from your body.

The frequency bands and transmitting power (radiated and/or conducted) nominal limits applicable to this radio equipment are as follows: MLY-B10/MLY-B11: Bluetooth: 12 dBm, NFC: 42dB μ A/m@10 m, WPT: 100 kHz-119 kHz, 135 kHz-140 kHz: 42dB μ A/m@10 m, 119 kHz~135 kHz: 66 descending 10 dB/dec above 0, 119 MHz@10m, 140 kHz-148.5 kHz: 37.7dB μ A/m@10 m.

Accessories and Software Information

It is recommended that the following accessories should be used:

Charging cradle: CP81-1 Wrist circumference measuring tape Independent cover

The product software version is 2.1.0.127

Software updates will be released by the manufacturer to fix bugs or enhance functions after the product has been released. All software versions released by the manufacturer have been verified and are still compliant with the related rules.

All RF parameters (for example, frequency range and output power) are not accessible to the user, and cannot be changed by the user.

Please go to **Settings > About > Regulatory information** on the device to view the E-label screen.

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Copyright © Huawei 2022. Alle Rechte vorbehalten.
Copyright © Huawei 2022. Kaikki oikeudet pidätetään.
Copyright © Huawei 2022. Wszelkie prawa zastrzeżone.
Copyright © Huawei 2022. Toate drepturile rezervate.
Huawei Device Co., Ltd. Çin'de üretilmiştir.
Huawei Telekomünikasyon Dış Tic. Ltd. Şti.